

Uniquely Designed for Simple, Accurate, and Secure On-Site Drug Screening

Safe

Self-enclosed system protects test administrator from contact with urine.

Accurate

97% - 99% correlation to GC/MSFDA Cleared, and standardized to SAMHSA cutoff levels.

Leak Proof

Internal threaded "thermos" cap eliminates spills or leaks.

Defensible

Split specimen testing eliminates potential for specimen contamination.



Controlled

Integrated but separate Status Stik allows you to control when and where test is performed.

Customizable

Extensive Status Stik test menu allows you to choose which drugs to test.

Secure


Three Tamper Evident Seals ensure chain of custody throughout the testing process.

Convenient

Donor friendly design with internal splash guards provide for sanitary specimen collection.

Split Specimen Testing Maintains Specimen Integrity From Collection to Confirmation

Maintaining specimen integrity throughout the testing process is crucial to providing accurate, defensible results that will withstand potential court challenges. This includes adhering to the Federal Guideline (SAMHSA) mandating split specimen collection and testing. This protocol involves the splitting of the collected urine specimen into 2 separate components A and B. One component (A) will be used during the initial screening test while component (B) will be sent to a laboratory for confirmation testing should the initial screen be positive. Component B must maintain

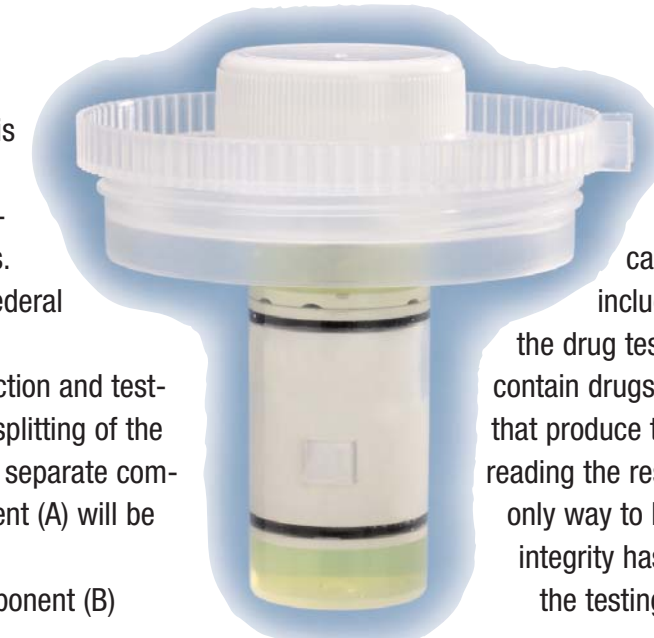

Status Cup Plus automatically splits the specimen into 2 separate chambers by using a patent pending one way valve system that is engaged by depressing the Status Stik. Once engaged, this valve seals off the main cup chamber from the test chamber.

specimen integrity and not be exposed to any potential source of contamination. Sources of contamination that can create "reasonable doubt" can include disposable pipets and possibly the drug test itself as these tests actually contain drugs conjugated to dye complexes that produce the colored lines you see when reading the result. Split specimen testing is the only way to be 100% sure that specimen integrity has been preserved throughout the testing process. It takes only one "questionable" result to have a significant impact on your company and your entire drug testing program.

PROCEDURE

- 


COLLECT... Record donor information on cup label. Have donor provide a minimum of 45ml of urine directly into cup (up to the minimum fill line on cup). Replace cap and return to test administrator.
- 


SECURE... "Lock" Status Cup closed using the built in security seal located on the Status Cup cap. Record temperature strip reading.
- 


SPLIT & TEST... Remove test well cap. Insert the Status Stik into the test well. Push down to depress internal valve that automatically splits the specimen and starts the test. You will observe the migration of the specimen through the Status Stik result window.
- 


READ RESULTS... Read results in 5 minutes.
Negative - visible line for each drug tested.
Positive (non-negative) - no visible line for each drug tested.
- 


CONFIRM... To perform GC/MS confirmation on positive results (non negatives), remove the Status Stik and replace test well cap. Send the Status Cup to an appropriate reference laboratory for confirmation.

More Than Just a Drug Test

LifeSign offers an extensive array of customer support programs designed to assist you in developing and managing your drug test program.

- **On-Site Support** from LifeSign's team of Substance Abuse Specialists to train, consult and offer program management solutions. Extensive training materials are available to assist you with chain of custody issues, legally defensible results and state regulations.
- **Technical Service Staff** is just a phone call away to provide immediate answers to your questions regarding test results, cross reactivity or drug pharmacology.
- **GC/MS Confirmations** by labs affiliated with LifeSign can reduce overall program costs, provide easier access and reduce administrative requirements. The cost of GC/MS can be a la carte or built into the price of each Status Cup Plus test.
- **MRO Services** can be recommended from a network of affiliated Medical Review Officers.
- **Cooperative Marketing Programs** to help your employees and customers realize the benefits of on-site drug testing.
- **Instant Off-Site Results.** Not ready to bring your drug testing in house, LifeSign can recommend local clinics using Status Cup Plus that can usually turnaround screening results in less than 60 minutes.
- **Turnkey Program Management** of your drug test program including drug testing, GC/MS confirm, MRO review, data management, and background screening is available from LifeSign's workplace partner Choicepoint®.

ORDERING INFORMATION (35 Status Stiks and Status Cups per box)

2 Panel	3 Panel	4 Panel	5 Panel
THC/COC- 72135	THC/COC/OPI- 73035	THC/COC/OPI/AMP- 74035	THC/COC/OPI/AMP/PCP- 75035
	THC/COC/MET- 73135	THC/COC/OPI/MET- 74135	THC/COC/OPI/MET/PCP- 75135

LifeSign's extensive menu of other substance abuse products lets you customize your substance abuse program.



STATUS STIK




STATUS DS



ADULTERATION STRIPS



STATUS ALCOHOL

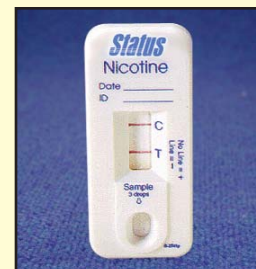

STATUS NICOTINE

Drug Testing Pacific Ltd. GPO Box 5066 Whangarei 0101 NZ
Tel. +64 9434 4669, Fax +64 9434 4669, Email email@drugtesting.net.nz

Introducing...Drug Testing Done Right.

Status Cup Plus

DRUG SCREENING SYSTEM



Split
Specimen
Testing
Without
Urine
Handling

Safe Secure Defensible